

Α ΕΛΛΗΝΙΚΗ ΔΗΜΟΚΡΑΤΙΑ  
ΥΠΟΥΡΓΕΙΟ ΥΓΕΙΑΣ  
ΕΘΝΙΚΟ ΚΕΝΤΡΟ ΕΡΕΥΝΑΣ ΔΙΑΓΝΩΣΤΙΚΗΣ ΜΙΚΡΟΒΙΟΛΟΓΙΑΣ ΚΑΙ ΜΟΝΙΤΟΡΑΣΙΑΣ

### CRP Uhs - UNIVERSAL APPLICATION

Diagnostic reagent for the quantitative in vitro determination of CRP (C – Reactive Protein) in human serum or plasma on photometric systems by turbidimetric immunoassay

**REF**

**Cont.**

- D04510 2 x 25 ml CRP Uhs Latex Reagent  
2 x 25 ml CRP Uhs Buffer
- D04561 5 x 1 ml CRP Uhs Cal. Series (for universal range)
- D99557 2 x 2 ml CRP Uhs Control Set (for universal range)

#### 1. REAGENT PREPARATION

The reagents are ready to use  
Reagent: CRP Uhs Buffer Start R1: CRP Uhs Latex Reagent

#### 2. Instrument setting

Item Name	CRP
Units	[mg/l]
Decimals	[2]
Calibration Type	[Spline]
1	[0.0 ]
2	[Cal 1 ]
3	[Cal 2 ]
4	[Cal 3 ]
5	[Cal 4 ]
6	[Cal 5 ]
Method Type	[END]
MainW. Length1	[505]
Subw. Length2	[700]
Method	[ ]
N-Range Male and Female	[#] – [#]
Reagent Kind	[Double]
Sample Volume	[3 µl]
Reagent 1 Volume	[150 µl]
Reagent 2 Volume	[150 µl]
Co-ordinate	Y = [1.000] X + [0.000]
Monitor Zero level point	[1]
Monitor Span	[3.000]
Third mix	[Off]
Water Blank	[On]

Read Main (Start-End)	[42] – [52]
Sub	[ ] – [ ]
Absorbance Limit	[-0.100] – [2.500]
Linear Check	[10%]
Endpoint Limit	[ ]
Factor: Blank Correction	[1.000]
Prozone	
Read	1 [ ] – [ ] 2 [ ] – [ ] 3 [ ] – [ ]
Limit	2 [ ] – [ ] 3 [ ] – [ ]
Auto Rerun SW	[Off]
Range (Result )	
Low-SW	[Off]
Serum	[#]
High-SW	[Off]
Serum	[#]
Absorbance	
Low-SW	[Off]
High-SW	[Off]
Prozone-SW	[Off]

- #) Data entry by the user
- \*) Input Standard or Calibrator value

333 Tests/Kit

www.trionmed.ru  
- www.trionmed.com  
Dialab - www.kuban-pm.ru

**Note:** These suggested instructions and instrument parameters are to be used in conjunction with the reagent package insert and the instrument operation manual. Refer to these documents for complete instructions before performing tests.